


The MFS Smart Catheter is designed to provide a continuous flow of anesthetic medication at a highly uniform level all along the surgical wound site. The Smart Catheter provides physicians with the ability to determine the actual length of the infusion segment.

THE Smart Catheter INFUSION TUBING

There are 15 marks on the catheter infusion segment (see below). The wide mark indicates the beginning of the segment and must be positioned just beneath the surface of the skin at the insertion site.

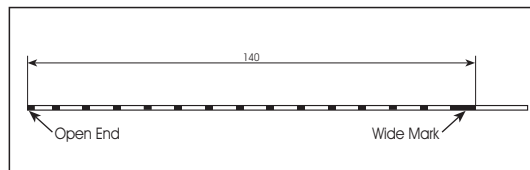

United State federal law restricts this device to sale to or on the order of a physician

The last mark is located at the open end of the catheter. The remaining 13 marks are placed 10 mm apart.

The procedure for determining the actual and appropriate length of the infusing tubing is very simple.

1. Prior to inserting the catheter, determine the length of the infusing tubing segment.
2. Using a pair of scissors, cut the segment of the tubing to the desired length by cutting the appropriate mark (as shown on the opposite page).

CAUTION

- Use aseptic technique.
- Refer to and follow the instructions for catheter placement described in SmartInfuser PainPump Physician Guidelines and Instructions.

United State federal law restricts this device to sale to or on the order of a physician